




**JDENTALCARE**  
*just smile*


**FDA**  
Approved



MADE IN ITALY

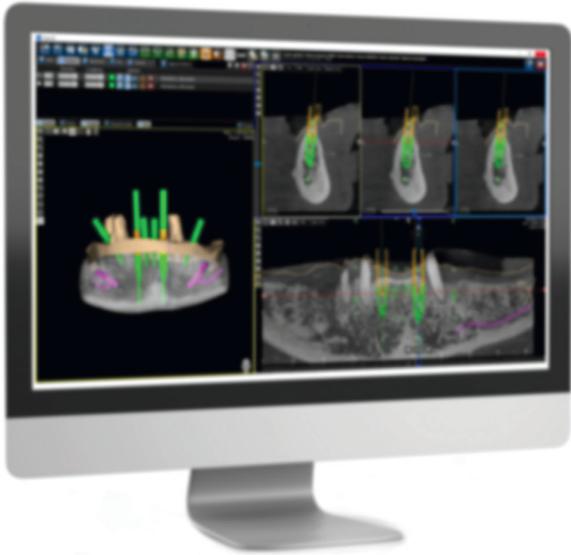

New design  
for better surgery

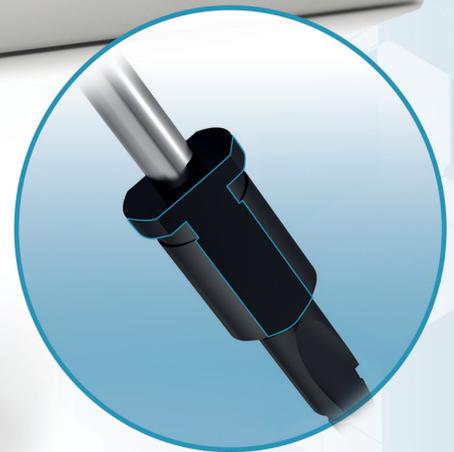


# JD Guided Surgery Kit

Friend of your hand.

# Simplicity

## One kit for all JD implant lines.


Simplified insertion with just one kit for all JD implant lines from Ø2.75 to Ø5.0 mm and from length 6 to 15 mm.

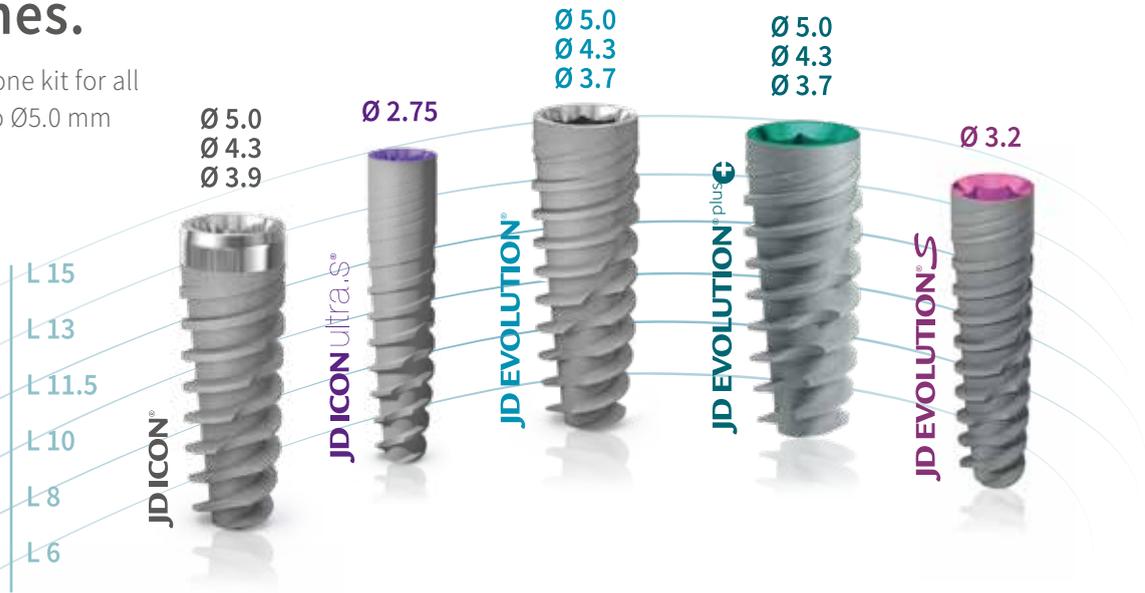

# Compatibility

## User-friendly software design.

A single kit compatible with the main software programs for guided surgery present on the market.

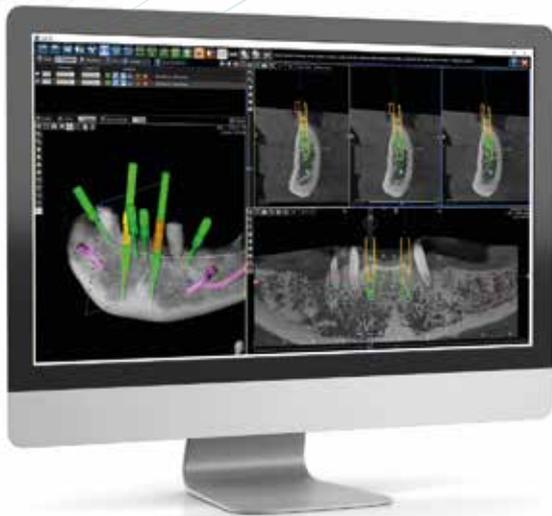

# Precision

## Guided implant depth control.

High precision in surgical procedures: implants are placed to the planned depth using a surgical guide and the guided surgery instruments are used in combination with a single sleeve of Ø5.05 mm for a secure implant depth control.



# Let your hand be guided by the JD Guided Surgery Kit.

The ideal solution to perform a minimally invasive surgery with more precision, increased predictability results and with few simple guided surgery steps.



## New design for better surgery

All JD Guided Drills have a non cylindrical special design with two lateral cuts which allows:

- a less friction on the sleeves, avoiding overheating
- a better external irrigation during the implant site preparation.

## Drilling protocols.

It is recommended to adhere to the indications of the following drilling sequence to ensure optimal primary stability of the implant.




### Attention:

In order to perform a correct osteotomy, the first drill that must be used is the drill Ø2.4 mm L 6.0 mm. This drill is designed also to flat the bone.

IMPLANT DIAMETER	SOFT BONE TYPE IV	MEDIUM BONE TYPE II-III	DENSE BONE TYPE I
Ø 2.75	2.0	2.0 2.4	2.0 2.4 2.8 L 6
Ø 3.2	2.0 2.4	2.0 2.4 2.8 L 6	2.0 2.4 2.8 3.2 L 6
Ø 3.7/3.9	2.0 2.4 2.8 L 6	2.0 2.4 2.8 3.2 L 6	2.0 2.4 2.8 3.2 3.6 L 6
Ø 4.3	2.0 2.4 2.8 3.2 L 6	2.0 2.4 2.8 3.2 3.6 L 6	2.0 2.4 2.8 3.2 3.6 4.2 L 6
Ø 5.0	2.0 2.4 2.8 3.2 3.6 L 6	2.0 2.4 2.8 3.2 3.6 4.2 L 6	2.0 2.4 2.8 3.2 3.6 4.2 L 8

**Note:** All dimensions are expressed in millimeters.

# Products.

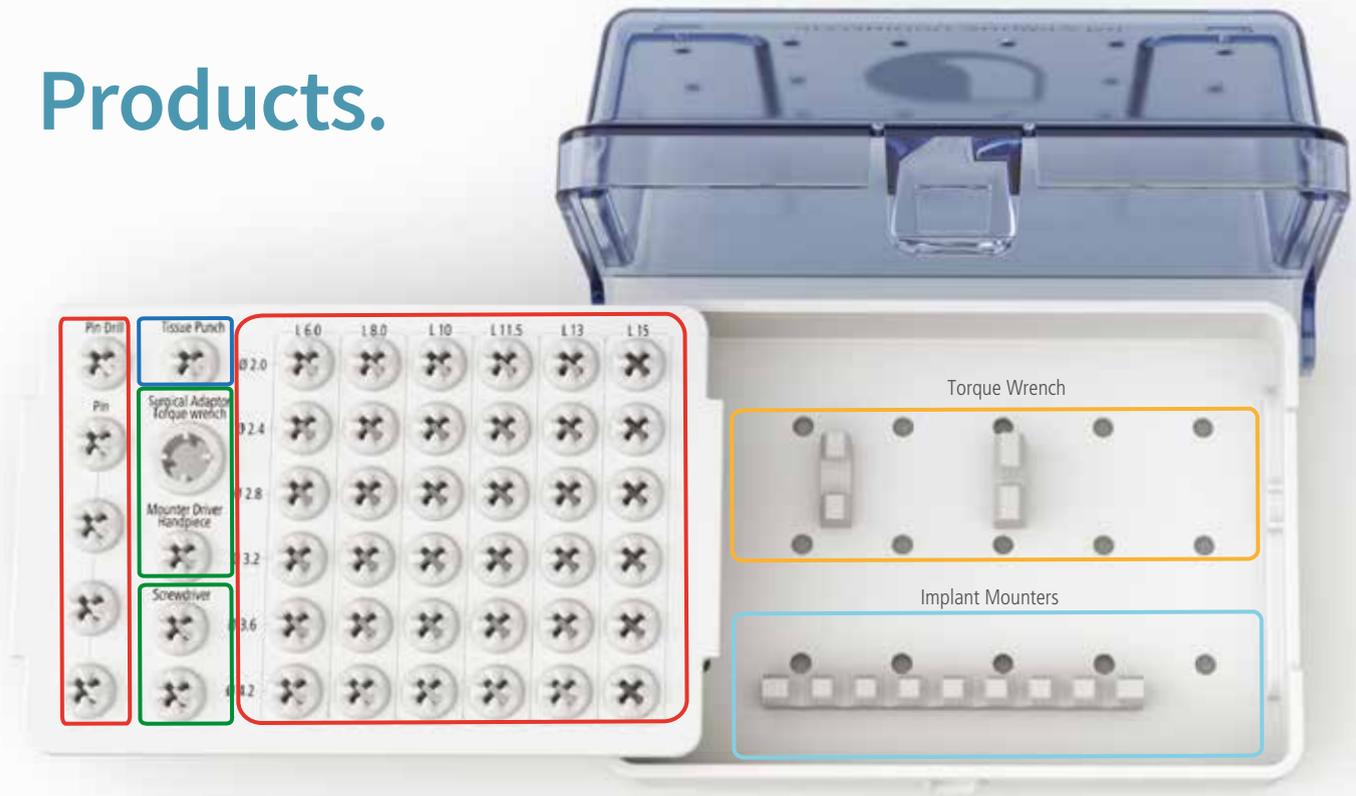


## JD Torque®



## Implant and Prosthetic Drivers




## Tissue Punch

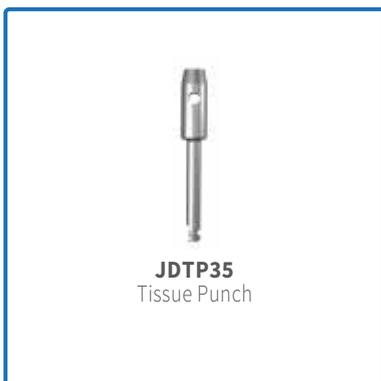

## Implant Mounters



## Guided Drills

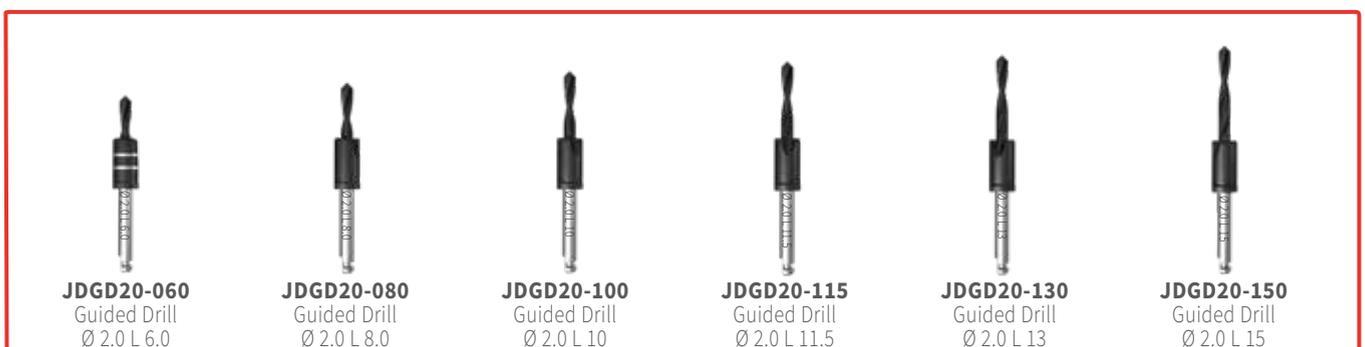

# Guided Drills



**JDGD24-060**  
Guided Drill  
Ø 2.4 L 6.0



**JDGD24-080**  
Guided Drill  
Ø 2.4 L 8.0



**JDGD24-100**  
Guided Drill  
Ø 2.4 L 10



**JDGD24-115**  
Guided Drill  
Ø 2.4 L 11.5



**JDGD24-130**  
Guided Drill  
Ø 2.4 L 13



**JDGD24-150**  
Guided Drill  
Ø 2.4 L 15



**JDGD28-060**  
Guided Drill  
Ø 2.8 L 6.0



**JDGD28-080**  
Guided Drill  
Ø 2.8 L 8.0



**JDGD28-100**  
Guided Drill  
Ø 2.8 L 10



**JDGD28-115**  
Guided Drill  
Ø 2.8 L 11.5



**JDGD28-130**  
Guided Drill  
Ø 2.8 L 13



**JDGD28-150**  
Guided Drill  
Ø 2.8 L 15



**JDGD32-060**  
Guided Drill  
Ø 3.2 L 6.0



**JDGD32-080**  
Guided Drill  
Ø 3.2 L 8.0



**JDGD32-100**  
Guided Drill  
Ø 3.2 L 10



**JDGD32-115**  
Guided Drill  
Ø 3.2 L 11.5



**JDGD32-130**  
Guided Drill  
Ø 3.2 L 13



**JDGD32-150**  
Guided Drill  
Ø 3.2 L 15



**JDGD36-060**  
Guided Drill  
Ø 3.6 L 6.0



**JDGD36-080**  
Guided Drill  
Ø 3.6 L 8.0



**JDGD36-100**  
Guided Drill  
Ø 3.6 L 10



**JDGD36-115**  
Guided Drill  
Ø 3.6 L 11.5



**JDGD36-130**  
Guided Drill  
Ø 3.6 L 13



**JDGD36-150**  
Guided Drill  
Ø 3.6 L 15



**JDGD42-060**  
Guided Drill  
Ø 4.2 L 6.0



**JDGD42-080**  
Guided Drill  
Ø 4.2 L 8.0



**JDGD42-100**  
Guided Drill  
Ø 4.2 L 10



**JDGD42-115**  
Guided Drill  
Ø 4.2 L 11.5



**JDGD42-130**  
Guided Drill  
Ø 4.2 L 13



**JDGD42-150**  
Guided Drill  
Ø 4.2 L 15



**JDGDP**  
Pin Drill



**JDPIN**  
Pin



**JDPIN**  
Pin



**JDPIN**  
Pin



**JDPIN**  
Pin

## Sleeves



**JDBGB**  
Template  
Sleeve in peek



**JDBG**  
Template  
Sleeve



**JDBGP**  
Pin Sleeve



**JDENTALCARE**

[www.jdentalcare.com](http://www.jdentalcare.com)